
movement and muscular contractions that promote the fecal mass ejection.

Instructions for use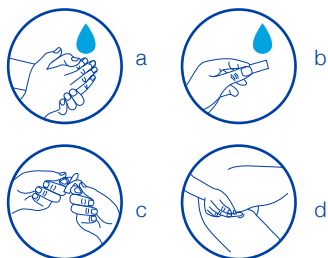

Accurately wash the hands before and after the use of suppositories (a). If the suppository is too soft put it for some minute in a refrigerator or under cold water (b). Remove the external aluminium (c) and wet the suppository with cold water or aqueous lubricant to facilitate the insertion. Lie down on the side and insert the sharp side of the suppository into the rectum. Gently push the suppository completely with a finger; otherwise it could be expelled (d). Remain lying down, possibly on your belly, for some minutes before to get up. SELLALAX should have effect within thirty minutes.

FOR RECTAL USE ONLY.

When and for how long use SELLALAX

It is advisable to use SELLALAX once a day. Consult your doctor in case of lack of results or if constipation persists. A prolonged use of laxatives (more than 7 consecutive days) is not recommended and could lead to addiction.

Excessive doses of the product can cause abdominal pain and diarrhea. Loss of liquids and electrolytes should be reintegrated.

Side effects

A mild rectal burning sensation could be perceived.

Recommendation for the maintenance of a correct intestinal functionality.

Drink much water, increase the fibres amount into the diet and regularly perform physical activity contribute to the functionality of the intestine.

Storage and disposal instructions

Keep the product in a cool and dry place, away from direct sunlight and source of heat. Do not store over 30°C. Do not dispose of in the environment after use.

Keep the product out from the reach of the children.

Shelf-life

The product has a 5 years shelf-life if correctly stored in its original container.

Packages and content

SELLALAX can be found in the following packages: box of 6, 12, 18, 30 or 50 suppositories.

Some packages could not be available.

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Glycerol suppositories

SELLA

Please read this leaflet carefully before using this product: it answers to some frequently asked questions on SELLALAX. Use the product according to the instructions provided in the leaflet and in case of doubts ask for clarification to your doctor or pharmacist.

Don't throw away this leaflet and keep it together with the product: it may be necessary to read it again.

Product presentation

SELLALAX is a disposable medical device for rectal use in form of suppository, recommended to facilitate the evacuation activity of the bowel.

Composition

One suppository of 2500 mg contains: glycerol 2315 mg, sodium stearate and sodium bicarbonate.

When the use of SELLALAX is recommended

SELLALAX is recommended for treatment of short duration of occasional constipation. SELLALAX is a suppository with a temporary action, which is able to lubricate fecal mass to promote its ejection. Do not use SELLALAX for any other purpose. Do not ingest. Only for rectal use.

When the use of SELLALAX is not advisable

Do not use SELLALAX in presence of cramps, severe abdominal pain or of unknown origin and if the patient suffers from intestine inflammatory diseases, as ulcerative colitis or Crohn's disease. Avoid to use the product also in presence of nausea, vomit, blood into the faeces, anal rhagades, intestinal obstructions, Ileus (loss of intestinal muscular tone), low levels of potassium into the blood and severe state of dehydration.

Do not use SELLALAX in case of allergy or hypersensitivity to glycerol or on of the other components. Do not use after the expiry

date printed on the box and the aluminium or if the package is damaged.

Inform your doctor if:

- you notice the absence of any intestinal movement, even for few days, or a sudden change of the usual bowel habits, for more than two weeks;
- the product has been ingested by accident;
- undesirable effects occur;
- you are taking other medicines or medical devices.

Use in children

SELLALAX is not indicated for children under 12 years.

Use during pregnancy or breast-feeding

There are not known interactions with other active pharmaceutical ingredients, therefore, in the absence of clinical data and in case of concomitant use with other medicines or medical devices, it is recommended to consult your doctor before using SELLALAX.

Activity and properties

Glycerol-based preparations intended for rectal use show an evacuant activity without the need of product absorption by the intestinal mucosa.

Glycerol, indeed, when inserted into the rectum is capable to promote the evacuation by a physical mechanism which does not involve a systemic absorption.

Glycerol softens the fecal mass and lubricate rectum wall, causing intestinal